DE Läusekamm Gebrauchsanweisung.....	IT Pettinina Istruzioni per l'uso	2	29
EN Lice comb Instruction for Use	TR Bit tarağı Kullanma Talimatı	9	36
FR Peigne anti-poux Mode d'emploi	RU Гребень от вшей Инструкция по применению	15	42
ES Lendreras Instrucciones para el uso	PL Grzebień na wszy Instrukcja obsługi	22	48

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

1. Introduction	9	8. Operation	12
2. Included in delivery.....	9	9. Cleaning and maintenance.....	13
3. Signs and symbols	9	10. Frequently asked questions.....	13
4. Intended use.....	10	11. Disposal.....	13
5. Safety notes.....	10	12. Technical data	13
6. Device description	12	13. WARRANTY / SERVICE.....	14
7. Initial use.....	12		

1. Introduction

The Beurer HT 15 lice comb allows you to simply and cleanly rid the scalp of nits and lice. The lice comb does not use any chemicals, meaning it is particularly gentle on the scalp and is also suitable for small children.

How does the lice comb work?

When it comes into contact with a louse or nit, the lice comb emits a harmless electrical impulse that is imperceptible to humans. When combing the hair, the lice are struck by the electrically charged metal teeth and effectively removed. The lice comb therefore provides a simple, safe and effective method to remove lice.

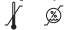
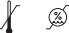
2. Included in delivery

- 1 x HT 15 lice comb
- 1 x protective cap
- 1 x cleaning brush
- 1 x AA 1.5V battery
- 1 x set of these instructions for use

3. Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and the accessories:

 WARNING	WARNING denotes danger with medium-level risk, which can lead to death or serious injury if ignored.
 CAUTION	CAUTION denotes danger with low-level risk, which can lead to minor or moderate injury if ignored.
NOTE	Note on important information
	Observe the instructions for use
	Application part, type B

	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Manufacturer
 PAP	Dispose of packaging in an environmentally friendly manner
	Batch designation
IP22	Protected against solid foreign objects 12.5 mm in diameter and larger, and against drops of water when the housing is angled up to 15°
Storage/Transport 	Permissible storage and transport temperature and humidity
Operating 	Permissible operating temperature and humidity
	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical devices.

4. Intended use

The device is only intended for combing human head hair, as well as cat and dog fur. The device is intended for private use only. This device may only be used for the purpose for which it is designed and in the manner specified in the instructions for use. Any form of improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or incorrect use.

5. Safety notes

WARNING

- Only use the device on dry, combed hair.
- Keep the device away from hot surfaces, and also protect it against direct sunlight.
- When not in use, always place the protective cap provided on the device.
- Operation in the vicinity of strong electromagnetic fields, for example next to a mobile phone, may cause malfunctions.
- Keep packaging material away from children. Danger of suffocation!
- Before use, inspect the device and accessories for visible damage. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.
- Do not use the device near water or in water.
- Never submerge the device in water or other liquids! Make sure that moisture does not get inside the device.
- Do not allow children to play with the device.
- Store the device out of the reach of children.
- This device may be used by children over the age of eight and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely and are fully aware of the consequent risks of use.
- Children must always be supervised by an adult during use of the device.

- After each use, clean the comb thoroughly using the cleaning brush – particularly before using the device on another person.
- During use, ensure that the comb does not come into contact with the mouth, eyes or ears.
- People with a sensitive scalp may experience slight tingling or itching during use. This is normal and not harmful to your health.
- Do not use the device on persons with heart conditions, epilepsy or pacemakers. People with heart conditions, epilepsy or pacemakers must also not operate the device on other people.
- Do not use the device on birthmarks or the scalp if the skin is broken.
- Skin contact with the metal teeth during use may result in painful but harmless electrical impulses.
- Do not touch the metal teeth when the device is switched on.
- The device complies with the EU Medical Devices Directive 93/42/EEC, the German Medical Devices Act (Medizinproduktegesetz) and the European standard EN 60601-1-2 (in accordance with IEC 61000-4-2, IEC 61000-4-3) and is subject to particular precautions with regard to electromagnetic compatibility. The device is suitable for use in all environments listed in these instructions for use, including domestic environments.

Notes on electromagnetic compatibility

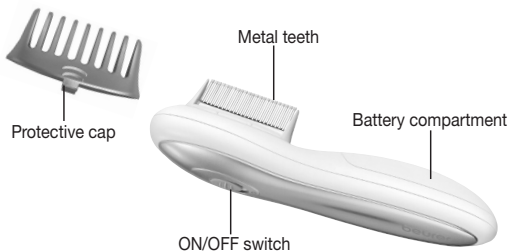
- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, rinse the affected area with water and seek medical assistance.
- **⚠ Choking hazard!** Small children may swallow and choke on batteries. Store batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
- **⚠ Risk of explosion!** Do not throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, open or crush the batteries.

6. Device description

7. Initial use

Inserting the battery

1. Slide the battery compartment lid open.
2. Insert the battery included in delivery into the battery compartment as shown inside the battery compartment. When inserting the battery, ensure that the correct polarity (+/-) is observed.
3. Then slide the battery compartment lid closed until you hear and feel it click into place.

8. Operation

Only use the lice comb on dry hair. Before use, comb the hair to be treated with a normal comb right to the tips until all tangles are removed.

1. Using your thumb and index finger, press on the notches on the side of the protective cap until it is released. Pull the protective cap up and off.
2. Move the ON/OFF switch to "I". A high-pitched beep sounds, which indicates that the device is switched on and ready for use.
3. Hold the lice comb at an angle of 45° to the scalp. Slowly move the lice comb from the roots to the tips of the hair. Slowly comb through all of the hair carefully and completely.
4. As soon as a louse comes into contact with the metal teeth of the lice comb, the beep audibly changes. If the high-pitched beep does not sound again, clean the metal teeth with the cleaning brush.
5. To end the application, move the ON/OFF switch to "0". The high-pitched beep is switched off.
6. Clean the lice comb after every use as described in the "Cleaning and maintenance" chapter.

NOTE

The lice comb removes hatched lice as well as unhatched nits (the eggs of the lice). Repeat the application every day for two weeks so that the lice are completely eliminated. In order to prevent lice returning, we recommend repeating the application once or twice a week for a further month. This means that any remaining lice can be removed.

9. Cleaning and maintenance

CAUTION

- Remove the protective cap for better cleaning of the metal teeth. Carefully clean the metal teeth after every use using the cleaning brush provided. Check the lice comb for any remnants between the metal teeth.
- Clean the device (apart from the metal teeth) after use with a soft, slightly damp cloth. If it is very dirty, you can also moisten the cloth with a mild soapy solution. Do not use any abrasive cleaning products.
- Ensure that no water enters the device. If this should happen, remove the batteries and only use the device again when it is completely dry.
- Protect the device from impacts, moisture, dust, chemicals, marked temperature fluctuations and nearby sources of heat (ovens, heaters).

10. Frequently asked questions

Who can be affected by head lice?

Anyone can be affected by head lice, not only children. Having a case of lice is also not caused by a lack of hygiene.

How do you get head lice?

Lice cannot fly or jump; they scuttle and crawl. Lice are transferred by direct contact with a person that already has lice (for example when two heads come into contact).

Where do head lice come from?

School is usually not the source of transfer. Lice are found much more often in communities outside of school.

Who should be checked and how?

If you have been informed of lice in your community, check all members of your family at least once a week and inform your friends.

Why is it important to ascertain whether lice are present immediately?

If you find out promptly that lice are present, it is easier to remove them as their numbers will be lower. There is also a lower probability that the lice will be transferred to other family members or acquaintances.

11. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers.

You are legally required to dispose of the batteries.

The codes below are printed on batteries containing harmful substances:

Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.

12. Technical data

- | | |
|----------------------------------|--------------------------------------------|
| • Model: HT15 | • Weight: approx. 56 g (without batteries) |
| • Battery: 1 x 1.5V AA batteries | • Dimensions: 158 x 37 x 50 mm |

13. WARRANTY / SERVICE

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.
German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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